

# Fentanyl Test Strips

## Purpose

BTNX fentanyl test strips version 2.0 (FTS) are immunoassay strips that detect the presence of fentanyl in drug samples. FTS are a useful tool in drug checking services because they can detect fentanyl and common fentanyl analogs in concentrations below the limit of detection of FTIR spectroscopy.<sup>1-3</sup> [Results from an FTS evaluation](#) conducted by the BCCSU, in collaboration with Health Canada (DAS), on samples collected in B.C. demonstrated high sensitivity to trace amounts of fentanyl and common fentanyl analogs.<sup>1</sup> Substances such as methamphetamine, MDMA, and diphenhydramine no longer cross-react with the latest version of FTS.<sup>1</sup>


## Scope

Below is the step-by-step procedure for using FTS as part of FTIR drug checking services. A combined test strip procedure is recommended for an efficient workflow that minimizes the amount of drug required for testing. Additional test strip procedures, including benzodiazepine test strips ([BTS](#)), medetomidine test strips ([MTS](#)), and xylazine test strips ([XTS](#)) are available on our [website](#).

This procedure is intended for drug checking staff using test strips, who are expected to be familiar with [safe handling](#), [drug crushing](#), and [disposal](#) procedures prior to use.

## Understanding Test Strips

When the test strip is dipped into a solution, the liquid moves up the strip and passes through a band of antibodies, which are proteins that bind to target substances. The antibodies are dyed, and as they move up the strip, they bind to the control and test lines, as shown below. The control line must appear for the test to be valid. However, if the target substance (i.e., fentanyl) is present, the antibodies will attach to it instead, and the test line will not appear.<sup>4</sup>

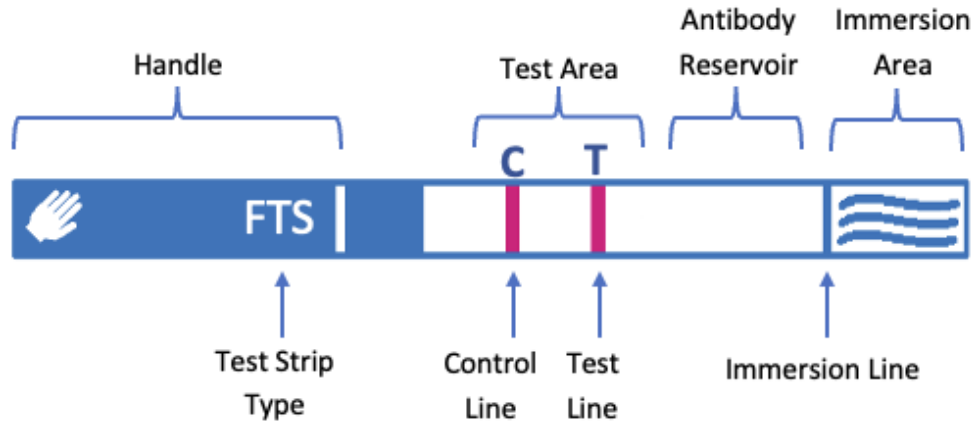


## Setup

Before starting, ensure a clear work area on an impermeable surface where the technician will not be bumped by passersby. Gather the following supplies:

PPE	Cleaning Supplies	Testing Equipment
<ul style="list-style-type: none"> <li>Nitrile gloves</li> <li>Face mask</li> </ul>	<ul style="list-style-type: none"> <li>Alcohol swabs</li> <li>Paper towels</li> </ul>	<ul style="list-style-type: none"> <li>1.5 - 2 ml microcentrifuge tube or paper cup</li> <li>Microscoop (in testing package)</li> <li>Drug crushing device</li> <li>Tap water</li> <li>Fentanyl test strip</li> </ul>

Check that the test strip has been stored correctly, the package is not punctured, and the expiry date has not passed (example below). Do not use damaged, opened, or expired test strips, as their results may be invalid.

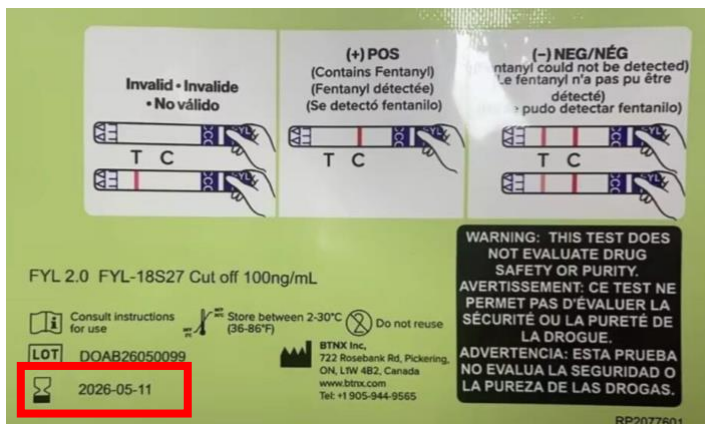

Expiry date on the test strip package.



Microcentrifuge tubes.

## Procedure

### Preparation

1. Put on disposable gloves and (optional) face mask.
2. Ensure the sample is thoroughly mixed. If not, [crush sample according to procedure](#).


### Prepare Solution

Microcentrifuge Tube	Paper Cup
3. Add <b>1-2 mg</b> of sample to the tube	Add <b>5-10 mg</b> of sample to the cup
4. Add <b>1 ml</b> of water to the same tube	Add <b>5 ml</b> of water to the same cup
5. Close and shake for <b>30 seconds</b> to dissolve	Mix with microscop to dissolve

*Note: The microcentrifuge tube is recommended for co-testing strips.*

### Test Sample

6. Dip the FTS to the immersion line (right-diagram) and absorb for **15 seconds**.
7. Set the FTS down on a non-absorbent surface (tip: use the packaging). Be mindful of cross-contamination. Wait **2 minutes** before reading the results.
8. Assess test strip under a direct, bright light. If the result appears positive at first, wait a full 5 minutes in case a faint test line shows. Do not interpret after 10 minutes (invalid).

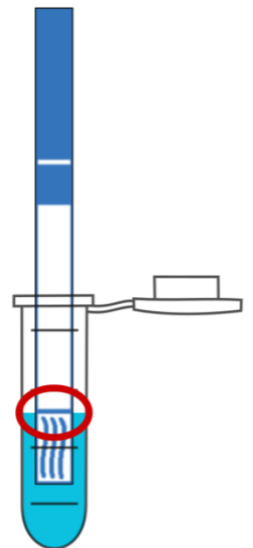

### Clean Workspace

9. [Dispose of solution](#) in clumping cat litter or an activated charcoal disposal pouch.
10. Clean work area and tools using isopropyl alcohol wipes, or if available, follow your worksite's cleaning protocols.
11. Dispose of all testing materials, including the used test strip and gloves.

## Interpreting Test Results

### 1. Positive Result

A positive test strip result (control-line only) indicates that fentanyl is **detected**.<sup>5</sup>



*Considerations:*

- The substance detected may be fentanyl or an analog.
- If result is unexpected, consider repeating the test to confirm (e.g., test-positive stimulant; dilute concentration and re-test).

### 2. Negative Result

A negative test strip result (both lines) indicates that fentanyl is **not detected**.<sup>5</sup>




*Considerations:*

- A faint test line still means a negative result, and does not indicate a low concentration of fentanyl and/or fentanyl analogs.
- The portion of the sample tested with FTS may not contain fentanyl or an analog, but the larger sample may (aka. “hot-spots”). See the [chocolate chip cookie effect](#).
- The sample tested may not have been crushed and mixed thoroughly enough to fully dissolve in the solution. See the [drug crushing procedure](#).
- Fentanyl or an analog may still be present below the [FTS detection threshold](#).
- If result is unexpected, consider repeating the test to confirm (e.g., expected fentanyl but test-negative; increase concentration and re-test).

### 3. Invalid Result

If the control line does not appear, the test is **invalid**.<sup>5</sup>

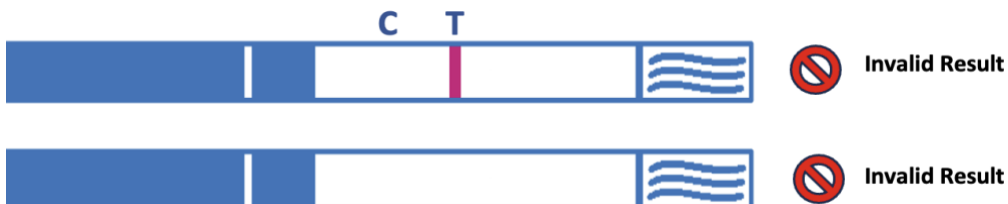

*Invalid test results can occur from:*

- Testing substances that are not compatible with test strips (e.g., alcohol).
- Saturating the result area (above the immersion line).
- Damaged test strip packaging, introducing humidity.
- Damaged test strip due to bending.
- Improper handling or storage.
- Expired test strips.

*Re-testing:*

1. Repeat the test using an unexpired test strip from an intact package.
2. Carefully conduct the test to rule out user error.
3. Ensure the test strip is being interpreted according to package instructions.

## Unexpected Performance

Test strips can sometimes perform unexpectedly, providing results that are either unlikely or known to be false. This can be due to unknown substance cross-reactivity or manufacturing errors that may affect a number of test strips in a batch (“bad-batch”). Re-test at least once with a new test strip if the result is unexpected. If the unexpected result is consistent, then document as follows:

1. Take a photograph of the used test strips with unexpected results.
2. Record the 10-character lot code(s) (e.g., DOA2204278).
3. Write a brief description of how the test strip performance was unexpected.
4. Send the above items in an email to notify BTNX ([support@btnx.com](mailto:support@btnx.com)) and BCCSU Drug Checking Program ([drugchecking@bccsu.ubc.ca](mailto:drugchecking@bccsu.ubc.ca)).

## Additional Resources

- [BTNX Rapid Response Fentanyl Test Strips \(Liquid/Powder\) – 100 Tests](#)
- [BTNX Manufacturer’s Instruction \(Fentanyl Test Strip – 2.0\)](#)
- [Government of Canada: Fentanyl](#)

## References

1. Angelucci J, Ruiz-Orduna A, Matthews J, Lessard S. Evaluation of BTNX Reformulated Fentanyl Test Strips [Internet]. British Columbia Centre on Substance Use; 2026 Mar. Available from: <https://drugcheckingbc.ca/wp-content/uploads/sites/4/2026/03/Evaluation-of-BTNX-Reformulated-Fentanyl-Test-Strips.pdf>
2. McCrae K, Tobias S, Grant C, Lysyshyn M, Laing R, Wood E, et al. Assessing the limit of detection of Fourier-transform infrared spectroscopy and immunoassay strips for fentanyl in a real-world setting. *Drug and Alcohol Review*. 2020;39(1):98–102. doi:10.1111/dar.13004
3. British Columbia Centre on Substance Use. Drug Checking Limitations [Internet]. Available from: <https://drugcheckingbc.ca/wp-content/uploads/sites/2/2020/05/BCCSU-Drug-Checking-Spectrometer-Disclaimer.pdf>
4. Koczula KM, Gallotta A. Lateral flow assays. *Essays Biochem*. 2016 Jun 30;60(1):111–20. doi:10.1042/EBC20150012 PubMed PMID: 27365041; PubMed Central PMCID: PMC4986465.
5. BTNX. Rapid Response® Fentanyl Test Strip [Manufacturer Instructions] [Internet]. Available from: [https://strategicstrike.blob.core.windows.net/lochness-products/resources/FYL-18S7-100\\_FYL-18S7-10\\_\(2\)\\_Instructions-Trilingual-\(EN-ES-FR\)-Product%20Insert-565.pdf](https://strategicstrike.blob.core.windows.net/lochness-products/resources/FYL-18S7-100_FYL-18S7-10_(2)_Instructions-Trilingual-(EN-ES-FR)-Product%20Insert-565.pdf)

### Prepared by

**Nicholas Purcell**, Program Engagement Coordinator, BCCSU

### Contributors

**Jen Angelucci**, Provincial Coordinator, Data Mobilization, BCCSU

**David Byres**, Training Coordinator, BCCSU

### Reviewers

**Jana Baller**, Lead, Drug Checking, Fraser Health

**Antoine Marcheterre**, Lead, Drug Checking, Interior Health

### Publication Date

May 07, 2026.